



QUICK USER GUIDE

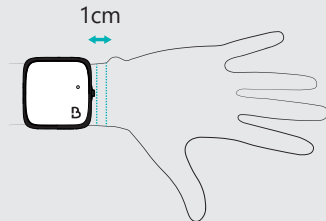

The Bora band® BB100 should fit snugly and apply light pressure to your skin, without causing any discomfort. This will ensure measurement accuracy.

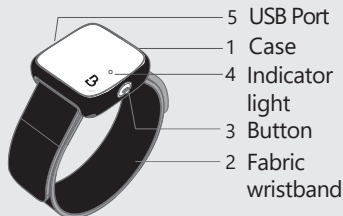



To start or check the operation of the Boraband®

Press the button: the fixed ● green indicator light means that the Bora band® is working.

> Good practice

- Wear your Bora band® **as often as possible**, including at night.
- Take advantage of your washing/shower time to **recharge the Bora band® daily**.
- Check that **your bracelet is properly connected** every day using the mobile application.

Connected oximeter & heart /
respiratory rate smart band monitor.

	Flashing orange indicator light: the battery is low, recharge your Bora band® BB100 using the charger provided.
	Flashing white indicator light: the battery is charging. The indicator light will turn off once the battery is fully charged.
	Blue: Bora band® is sending or receiving data; please do not switch off during this process.
	Fixed red indicator light: your Bora band® BB100 is faulty. <ul style="list-style-type: none">▶ Try to turn it off and then switch it on again.▶ If still faulty: contact the organisation that provided you with the Boraband® BB100.

Stopping the Boraband®

Press the button until the  green indicator light flashes.
This indicates that your Bora band® BB100 is switching off.

User ID:

Password:

Name and number of the HCP:

Name of the prescribing physician:.....

Manufacturer:



BIOSENCY
13 rue Claude Chappe – Bât A
Oxygène
35510 Cesson Sévigné
France
www.biosency.com

For more information,
please refer to the
Bora band® user manual.

