


QUICK USER GUIDE

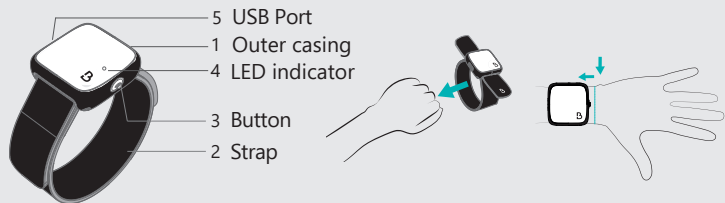

To guarantee the quality of measurements, the strap of the BORA Band BB100 should be tight but comfortable.

> Start-up

Push the button located on the side of the BORA Band. The indicator turns green ● during **5 seconds** to indicate that the BORA Band is up and running.

> Check proper functioning


Perform a short push on the button. The indicator turns green ● during **5 seconds** to confirm that the BORA Band is properly functioning.





> Switch-off

Ensure that your BORA Band has transmitted all acquired measurements before switching it off.

To do so, ensure that your smartphone is connected to the Internet, and that your BORA Band is in the vicinity of your smartphone.

Then, log on to your BORA Connect account and verify that your latest measurements are displayed.

To switch off, push the button during 2 seconds. The LED indicator  blinks during 5 seconds and indicates that your BORA Band BB100 is switching off.

	Blinking orange : battery level is low; charge your BORA Band BB100 with the provided charger.
	Blinking white: your BORA Band BB100 is charging. Once the BORA Band is fully charged, the led indicator will switch off.
	Blue: BORA Band is transmitting or receiving data, do not switch it off.
	Red: your BORA Band BB100 has encountered an error ▶ Try restarting your BORA Band ▶ If the error persists: contact your health provider structure at the address below

Notes:

For more detailed information, refer to the user manual of the BORA Band



Manufacturer:



BIOSENCY

1137 A Avenue des champs blancs
35510 Cesson Sevigné
France
www.biosency.com